

Face Shield Instructions

(Link provided by Miami Valley Hospital)

Supplies Needed

8 ½ X 11 Transparency Sheet

Velcro tabs (can cut long Velcro into small tabs)

Elastic 3/8" – 3/4" in width cut into 14-15" lengths

Foam cut into 1 ½ x 9-inch pieces (I used 1" project foam, it's softer and more flexible.)

Glue gun (gorilla glue doesn't work as well)

Instructions:

1. Lay the transparency in the landscape position.
2. Place a Velcro tab in both top corners about ¼" from the edges. This will help hold the elastic in place.
3. Place 2 lines of glue in the upper half of the foam piece.
4. Place the foam piece between the 2 Velcro tabs glue side toward transparency, with glue nearest to edge of the transparency. Press firmly.
5. Next place a ½" circle of glue on one Velcro tab. Press the elastic into the glue and use tip of scissors to press elastic into glue. Allow to dry.
6. Repeat step 5 on other side with other end of elastic.
7. Allow to dry and set in place.

