
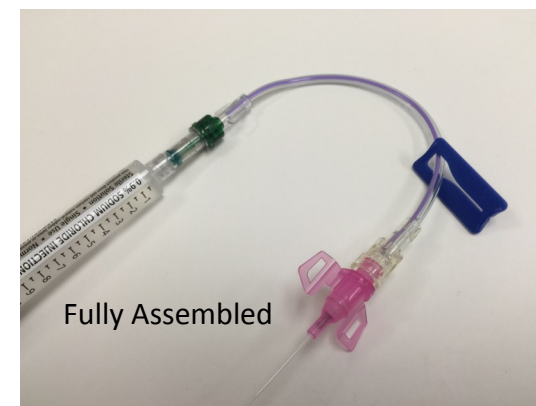


Quick Guide to Set Up New Braun IV Catheter With Locking Hub and J Loop

Set Up

1. Connect Green Locking Hub to J Loop. Attach Normal Saline Flush to Hub & Prefill the J Loop with Saline.
2. Start IV.
3. Connect J Loop to the IV Catheter.

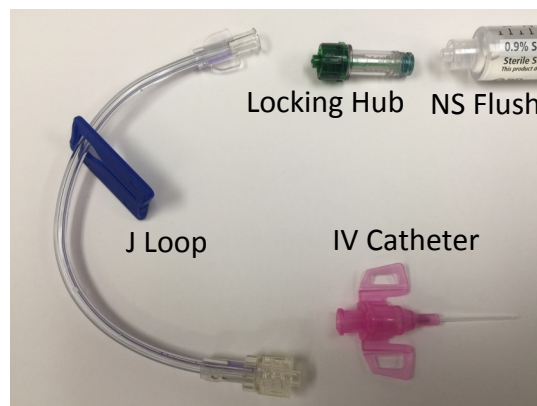

Trouble Shooting

Cross Threading

Turn NS Flush Counter Clockwise a Quarter Turn then Turn Clockwise Until Threads are Completely Flush with Hub.



Connecting Problem

Ensure to Depress NS Flush Firmly Against Hub Before Attempting to Twist.



EMS Center of Excellence

